

THANK YOU FOR USING THE VERTALOC® ACTIVE ANKLE BRACE

Your VertaLoc® ACTIVE Ankle Brace was designed to maximize support of your ankle while providing you with the comfort you need.

YOU HAVE RECEIVED:

- ONE (1) ankle brace with complete closure system.

FIT YOUR BRACE IN 3 EASY STEPS



1. OPEN ALL STRAPS AND POSITION THE FOOT UPRIGHT IN THE BRACE.



2. WRAP THE UPPER VELCRO BELT AROUND YOUR CALF.



3. WRAP THE BOTTOM VELCRO STRAP FROM RIGHT TO LEFT TO SECURE THE BASE TO THE ANKLE. MAKE SURE PRESSURE IS DISTRIBUTED EQUALLY AROUND THE CIRCUMFERENCE FOR BEST PERFORMANCE.

CARING FOR YOUR VERTALOC® ACTIVE ANKLE BRACE:

Hand wash fabric components in warm water using mild soap. Rinse thoroughly. Air dry out of direct sunlight.

- DO NOT STORE THE BRACE IN A HOT (80°F OR MORE) OR HUMID (80% OR MORE) AREA.
- STORE THE BRACE IN A COOL, DRY PLACE (HEAT MAY CAUSE BRACE DEFORMATION).
- KEEP THE BRACE AWAY FROM DUST OR POLLUTANTS.

VERTALOC® ACTIVE ANKLE BRACE WARRANTY:

- ✓ YOUR VERTALOC® ACTIVE ANKLE BRACE IS WARRANTED TO BE FREE OF MATERIAL OR WORKMANSHIP DEFECTS.
- ✓ ANY DEFECTIVE COMPONENTS OF YOUR VERTALOC® ACTIVE ANKLE BRACE MAY BE REPLACED OR CREDITED AT THE DISCRETION OF VERTALOC.
- ✓ THIS PRODUCT IS WARRANTED FOR ONE YEAR FROM THE DATE OF PURCHASE.



QUESTIONS OR COMMENTS? WE ARE ALWAYS HERE TO HELP YOU.
CALL US TOLL FREE 855- VERTALOC (855-837- 8256)