

THANK YOU FOR USING THE VERTALOC® PERFORMANCE WRIST BRACE

(MODEL# VL307PWB)

Your VertaLoc® Performance wrist brace was designed to maximize stability of your wrist while providing support you need to be active.

YOU HAVE RECEIVED:

- ONE (1) WRIST ORTHOSIS COMPLETE WITH CLOSURE SYSTEM
- ONE (1) ALLEN KEY
- ONE (1) MESH BAG



FIT YOUR BRACE IN A FEW EASY STEPS



1. OPEN ALL STRAPS. OPEN FABRIC IN THE BACK OF THE BRACE TO ADJUST THE RANGE OF MOTION.

3. PLACE THE WRIST AROUND THE CONE AND FINGERS BETWEEN WEDGES.



2. USE THE ALLEN KEY TO LOCK THE DESIRED DEGREE OF FLEXION OR EXTENSION.

4. TIGHTEN ALL STRAPS UNTIL SNUG AND COMFORTABLE.



CARING FOR YOUR VERTALOC® PERFORMANCE WRIST BRACE:

Remove aluminum support before washing. Hand wash in warm water using mild soap. Rinse thoroughly by hand, line dry only. Do not machine wash or dry. Do not expose to direct heat sources such as heaters, stoves, and radiators.

VERTALOC® PERFORMANCE WRIST BRACE WARRANTY:

- ✓ YOUR VERTALOC® PERFORMANCE IS WARRANTED TO BE FREE OF MATERIAL OR WORKMANSHIP DEFECTS.
- ✓ ANY DEFECTIVE COMPONENTS OF YOUR VERTALOC® PERFORMANCE BRACE MAY BE REPLACED OR CREDITED AT THE DISCRETION OF VERTALOC, INC.
- ✓ THIS PRODUCT IS WARRANTED FOR ONE YEAR FROM THE DATE OF PURCHASE.

QUESTIONS OR COMMENTS? WE ARE ALWAYS HERE TO HELP YOU
CALL US TOLL FREE 855-VERTALOC (855-837-8256)