

THANK YOU FOR USING THE VERTALOC® LITE WRIST BRACE

(MODEL# VL302LW)

Your VertaLoc® LITE wrist brace has been designed to maximize support and enhance comfort while allowing breathability.

YOU HAVE RECEIVED:

- ONE (1) WRIST ORTHOSIS COMPLETE WITH CLOSURE SYSTEM



FIT YOUR BRACE IN 4 EASY STEPS



1. OPEN ALL STRAPS. INSERT YOUR HAND INTO THE BRACE.

2. FASTEN THE THUMB STRAP SECURELY. NEXT, FASTEN THE OTHER TWO (2) STRAPS BY WRAPPING EACH STRAP AROUND YOUR WRIST AND THROUGH THE "D" RING. FASTEN SECURELY.



3. TIGHTEN ALL STRAPS UNTIL SNUG AND COMFORTABLE.

4. TO IMPROVE COMFORT, TRIM STRAPS WITH SCISSORS.



CARING FOR YOUR VERTALOC® LITE WRIST BRACE :

Remove aluminum support before washing.

Hand wash in warm water using mild soap. Rinse thoroughly by hand, line dry only.

Do not machine wash or dry.

Do not expose to direct heat sources such as heaters, stoves, and radiators.

VERTALOC® LITE WRIST BRACE WARRANTY:

✓ YOUR VERTALOC® LITE IS WARRANTED TO BE FREE OF MATERIAL OR WORKMANSHIP DEFECTS.

✓ ANY DEFECTIVE COMPONENTS OF YOUR VERTALOC® LITE BRACE MAY BE REPLACED OR CREDITED AT THE DISCRETION OF VERTALOC, INC.

✓ THIS PRODUCT IS WARRANTED FOR ONE YEAR FROM THE DATE OF PURCHASE.

**QUESTIONS OR COMMENTS? WE ARE ALWAYS HERE TO HELP YOU.
CALL US TOLL FREE 855-VERTALOC (855-837-8256)**