

THANK YOU FOR USING THE VERTALOC® MAX OA KNEE BRACE

Your VertaLoc® MAX OA Knee Brace has been designed to maximize stability of your knee while providing you with the comfort you need.

YOU HAVE RECEIVED:

- ONE (1) KNEE BRACE WITH COMPLETE CLOSURE SYSTEM
- ONE (1) SET OF 7 PADS (PLEASE CONSULT WITH YOUR PHYSICIAN BEFORE USING PADS)



FIT YOUR BRACE IN 4 EASY STEPS



1. RAISE YOUR LEG AND INSERT INTO THE BRACE. ADJUST THE DEVICE SO THAT THE CENTER OF THE KNEE IS AT THE KNEE ANGLE CONTROLLER.

2. PULL AND ATTACH STRAPS.



3. ADJUST THE MAX OA KNEE BRACE BY TIGHTENING THE VELCRO FASTENING STRAPS IN THE FOLLOWING ORDER: 1→2→3→4→5→6 (FOR THE BEST FIT AND PERFORMANCE, THE BRACE SHOULD BE SECURELY FASTENED).

4. ONCE THE BRACE IS IN PLACE, SET THE DIAL TO CONTROL YOUR DESIRED RANGE OF MOTION.



CARING FOR YOUR VERTALOC® MAX OA: *Hand wash fabric components in warm water using mild soap. Rinse thoroughly. Air dry out of direct sunlight.*

STORING YOUR VERTALOC® MAX OA:

- DO NOT STORE THE DEVICE IN A HIGH-TEMPERATURE (80°F OR MORE), HIGH HUMIDITY (80% OR MORE) PLACE.
- STORE THE DEVICE IN A COOL PLACE (HEAT MAY CAUSE DEVICE DEFORMATION).
- KEEP THE DEVICE AWAY FROM DUST OR POLLUTANTS.
- DO NOT KICK OR THROW THE DEVICE.

VERTALOC® MAX OA WARRANTY:

- ✓ YOUR VERTALOC® MAX OA IS WARRANTED TO BE FREE OF MATERIAL OR WORKMANSHIP DEFECTS.
- ✓ ANY DEFECTIVE COMPONENTS OF YOUR VERTALOC® MAX OA MAY BE REPLACED OR CREDITED AT THE DISCRETION OF VERTALOC, INC.
- ✓ THIS PRODUCT IS WARRANTED FOR ONE YEAR FROM THE DATE OF PURCHASE.

QUESTIONS OR COMMENTS? WE ARE ALWAYS HERE TO HELP YOU.
CALL US TOLL FREE 855-VERTALOC (855-837-8256)