

THANK YOU FOR USING THE VERTALOC® PRO ANKLE BRACE

MODEL# VL305PAB

Your Vertaloc® PRO Ankle Brace was designed to maximize support of your ankle while providing you with the comfort you need.

YOU HAVE RECEIVED:

- ONE (1) ankle brace with complete closure system
- TWO (2) soft removable pads
- ONE (1) upper calf extension (removable)
- ONE (1) lower leg strap support (removable)



FIT YOUR BRACE IN 3 EASY STEPS



1. OPEN ALL STRAPS. INSERT YOUR FOOT THROUGH THE BACK SIDE OF THE BRACE. CLOSE THE BACK STRAPS. ATTACH THE UPPER CALF EXTENSION TO THE BACK OF THE BRACE.



2. CLOSE THE STRAPS IN THE FRONT. WRAP EACH VELCRO BELT AROUND YOUR FOOT.



3. PLACE THE LOWER LEG STRAP SUPPORT BEHIND THE HEEL WITH THE "BOTTOM" PART POSITIONED UNDER THE HEEL. WRAP THE VELCRO STRAP FROM RIGHT TO LEFT TO SECURE THE BASE TO THE ANKLE. MAKE SURE PRESSURE IS DISTRIBUTED EQUALLY AROUND THE CIRCUMFERENCE FOR BEST PERFORMANCE.

CARING FOR YOUR VERTALOC® PRO ANKLE BRACE:

*Hand wash fabric components in warm water using mild soap.
Rinse thoroughly. Air dry out of direct sunlight.*



DO NOT WEAR THE ANKLE BRACE ON SLIPPERY UNEVEN SURFACE.

- DO NOT STORE THE BRACE IN A HOT (80°F OR MORE) OR HUMID (80% OR MORE) LOCATION.
- STORE THE BRACE IN A COOL, DRY PLACE (HEAT MAY CAUSE BRACE DEFORMATION).
- KEEP THE BRACE AWAY FROM DUST OR POLLUTANTS.

VERTALOC® PRO ANKLE BRACE WARRANTY:

- ✓ YOUR VERTALOC® PRO ANKLE BRACE IS WARRANTED TO BE FREE OF MATERIAL OR WORKMANSHIP DEFECTS.
- ✓ ANY DEFECTIVE COMPONENTS OF YOUR VERTALOC® PRO ANKLE BRACE MAY BE REPLACED OR CREDITED AT THE DISCRETION OF VERTALOC.
- ✓ THIS PRODUCT IS WARRANTED FOR ONE YEAR FROM THE DATE OF PURCHASE.

QUESTIONS OR COMMENTS? WE ARE ALWAYS HERE TO HELP YOU.
CALL US TOLL FREE 855-VERTALOC (855-837-8256)